

GUIDE ON HOW TO CUSTOMIZE THE HEARING OF EACH EARBUD

Not everyone has hearing problem with both ears, so you should have the option to adjust the volume level of Left Earbud and Right Earbud individually by yourself without the doctor visit.

Customizing your hearing is now easier than ever with AMPSound. With a small screwdriver which is included in the box, you can increase or decrease the volume easily:



HOW TO USE

Increase volume

Remove the battery capsule and screw clockwise.



Decrease volume

Remove the battery capsule and turn counter-clockwise.



WARM REMIDNER

Turning the screw one time most likely won't result in the preferred volume level, so we recommend screwing it bit by bit and listening to it a few times in your ear to see if you've reached the desired volume.