
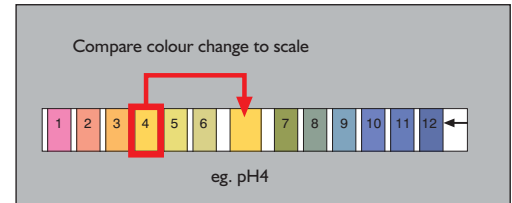



1

Draw up aspirate with a 60ml syringe and place on the pH papers reactive strip.
If the PH is 5 or below it is ok to feed.
If the PH is 6 or above then it is not ok to feed

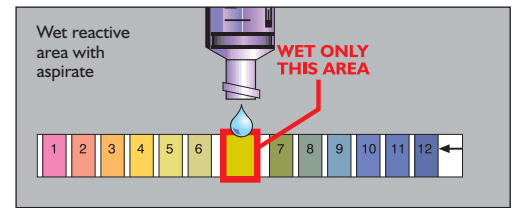


2

Flush the tube and connector threads with a 60ml syringe before feeding/delivery of medication. This prevents any blockages and any disconnection problems.

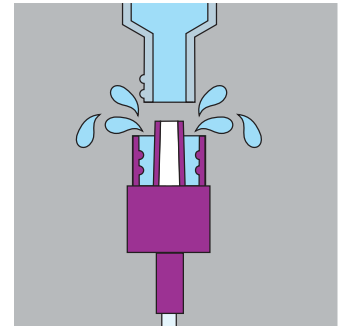

3

Administer feed. Ensure connections are not primed with feed as the threads can become clogged with feed and debris.

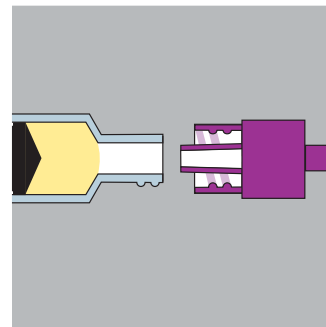

4

Flush the tube and connector threads with a 60ml syringe after feeding/delivery of medication. This prevents any disconnection problems and prolongs the life of the tube.

Medicina Ltd.

Units 1-4, Rivington View Business Park, Station Road, Bolton BL6 5BN, UK
Telephone: +44(0)1204 695050 email: info@medicina.co.uk

