



Platform plate fitting instructions

Please read these instructions fully before use. If in doubt about any aspect of this product or how to fit it then please contact your supplier.

Fitting

- 1) Bend hooks in centre of plate down 90 degrees.
- 2) Revolve plate so as the two small holes are at the same end as the backrest holes on the platform
- 3) Hook the plate onto the far side of the platform and allow the hooks to drop through the centre slots on the platform.
- 4) Slide the plate down the platform so the hooks engage and the backrest holes align.
- 5) Ease the plate over the edge of the platform starting at one end.

Removal

- 1) Lift the platform and starting at one end ease the lip of the plate off using two thumbs.
- 2) Slide the plate up the platform to disengage the centre hooks.
- 3) Unhook the plate from the far side of the platform.

Plate should be removed periodically for cleaning. This can be done with the same detergent as what is used to clean the bath.