

- Precleans Instrument and Endoscope Surfaces
- Promotes long lasting moisture retention on instruments and scopes for up to 72 hours
- Multi-Tiered Detergent Spray, Non-Aerosol

OneTAB PreSoak™ is a multi-tiered, neutral pH, humectant detergent spray that promotes water molecules into the soils leaving a hydrolysable web on surfaces to prevent soils from hardening for easy removal during the cleaning process

- Safe for all scopes & instruments
- Perfect for reprocessing delays
- Starts working instantly on contact
- No messy spills or splashing
- Environmentally friendly
- Ergonomic spray handle
- Neutral pH; Non-aerosol; Multi-tiered detergent
- Dual purpose; PreSoak and manual clean detergent

Directions for Use:

- Spray directly on all surfaces immediately after use.
- Actuate instruments during application.
- Reapplication is not necessary if all surfaces have been wetted.
- Once saturated, the soil can be removed any time up to 72 hours.
- Rinse after cleaning.
- If organic matter is difficult to remove during manual cleaning, extended soak time or cleaning may be necessary.

Flexible Endoscopes (Elevator Mechanism)

- After precleaning, spray OneTAB PreSoak directly into entire elevator mechanism area while slowly actuating the elevator mechanism up and down.
- Repeat manipulation of the elevator mechanism multiple times while applying OneTAB PreSoak.

Cautions:

Avoid prolonged contact with skin. Keep away from children. Do not swallow.



Item No. OTL-PS
Qty. CA/4