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M.O.COM. Srl
Via Saliceto, 22
40013 Castel Maggiore (BO)
Italy

website www.mocom.it

INSTRUMENT WASHER THERMO-DISINFECTOR

OPERATING MANUAL

ENGLISH

REVISIONS

The following table lists subsequent editions/revisions of the manual.
The "Description" field brief explains the subject of the latest revision.

Code	Rev.	Date	Description
D#0MWAA0000X	0	09-2005	First issue
D#0MWAA0000X	1	06-2007	Recycling and disposal instructions
D#0MWAA0000X	2	05-2008	ECC standard reference updating Additive option
97050465	3	05-2011	Address and code updating on Mocom use and maintenance manuals

TABLE OF CONTENTS

- GENERAL** 1
 - APPLICABLE EUROPEAN DIRECTIVES 1
 - INTENDED USE 1
 - MANUAL SCOPE 2
 - GENERAL WARNINGS 2
- CONTENTS OF THE PACKAGE** 3
 - DIMENSIONS AND WEIGHT 3
 - DESCRIPTION OF THE CONTENTS 3
 - LIFTING AND TRANSPORT 3
- PRODUCT INFORMATION** 4
 - INTRODUCTION 4
 - GENERAL FEATURES 4
 - FRONT VIEW 5
 - REAR VIEW 5
 - CONTROL PANEL 6
- INSTALLATION** 7
 - INTRODUCTION 7
 - Size and weight 7
 - Power supply 7
 - GENERAL PRECAUTIONS FOR THE INSTALLATION 7
 - COMPARTMENT DIMENSIONS FOR BUILT-IN INSTALLATIONS 8
 - ELECTRICAL CONNECTION 8
 - HYDRAULIC CONNECTION 9

Water supply connection	9
Water drain connection.....	9
EMBEDDED WATER SOFTENER	10
FILLING THE DETERGENT	10
FILLING THE ADDITIVE (OPTION).....	11
OPERATION.....	12
MACHINE STATUSES	12
LOAD THE MATERIALS.....	12
Handpieces and turbines	12
CHOICE THE CYCLE.....	13
START THE CYCLE	13
DRYING OFF.....	13
STOP TEMPORARY AND COMPLETELY THE CYCLE.....	13
END OF CYCLE	13
IN CASE OF BLACK-OUT	14
DOOR UNBLOCKING PROCEDURE.....	14
RESINS REGENERATION.....	14
APPENDIX A – TECHNICAL CHARACTERISTICS	15
SUMMARY TABLE	15
SAFETY DEVICES	16
RECOMMENDED DETERGENTS AND ADDITIVES	16
WARNINGS ABOUT THE CHEMICAL PRODUCTS	16
APPENDIX B – PROGRAMMING	17
ACCESSING THE MENU	17
Set the date and time.....	17
Select the language for the displayed messages	18
Print the stored reports	18
Deleting the stored Events History.....	19
APPENDIX C – MAINTENANCE	20
ORDINARY MAINTENANCE PROGRAM	20
MAINTENANCE BY THE OPERATOR.....	20
General warnings.....	20
Cleaning the external surfaces	21
Cleaning the control panel	21
Cleaning the nozzles	21
Disinfecting the external surfaces.....	21
Cleaning/replacing the handpiece and accessory holder.....	21
Regenerating the water softener	22
MAINTENANCE BY THE SERVICE	23
APPENDIX D – GENERAL PROBLEMS.....	24
INTRODUCTION	24
INTERNAL SELF DIAGNOSTICS.....	24
TROUBLESHOOTING AND SOLUTIONS.....	25
APPENDIX E – ALARMS INDICATIONS	26
INTRODUCTION	26
MEANING OF THE ALARMS ON THE DISPLAY.....	26
CODES REPORTED IN THE FIELD “FAULTS” OF THE EVENTS HISTORY	27
APPENDIX G – DECLARATION OF CONFORMITY	28
APPENDIX H – OPERATOR’S NOTES	29
APPENDIX Z – TECHNICAL SUPPORT.....	30

GENERAL

Symbols

APPLICABLE EUROPEAN DIRECTIVES

INTENDED USE

Important notes

Dear Customer

Thank you for choosing a product from M.O.COM. Srl. We hope that you will find it completely satisfactory.

This manual describes all procedures for the correct installation and use of the equipment.

Should you have any questions or suggestions concerning this product or our support, do not hesitate to contact us.

ENGLISH

NOTA

PAY SPECIAL ATTENTION; NOTES FOR PROPER INSTALLATION AND MAINTENANCE PROCEDURES AND USE METHODS.

WARNING

THIS SYMBOL INDICATES A POTENTIAL DANGER OF INJURY. FOLLOW THE PROCEDURES DESCRIBED IN THE MANUAL TO AVOID INJURING THE USER AND/OR OTHERS.

DANGER

THIS SYMBOL SIGNALS A POSSIBLE DANGER FOR PEOPLE. FOLLOW THE SUGGESTED PROCEDURE IN ORDER TO PREVENT POSSIBLE DAMAGE TO USERS AND/OR OTHERS.

DANGER

THIS SYMBOL INDICATES A POTENTIAL DANGER DUE TO HIGH TEMPERATURE.

THE MATERIAL THE EQUIPMENT IS COMPOSED OF MUST BE DISPOSED ACCORDING TO THE DIRECTIVE 2002/96/CEE.

The product described in this manual is manufactured in accordance with the highest safety standards and doesn't represent any danger for the operator if used according to the following instructions. The product is in accordance with the following European Directive as applicable:

2006/95/CE, for the approximation to the legislation of the Members States related to low voltage equipment.

2004/108/CE, for the approximation to the legislation of the Members States related to the electromagnetic compatibility;

93/42CEE and **subsequent changes**, concerning the medical devices.

This unit is designed for the washing and thermo-disinfection of instruments, trays and objects used in dental surgeries, hospital wards, clinics etc.

WARNING

THE UNIT MUST ONLY BE USED BY QUALIFIED PERSONNEL. IT MAY NOT BE USED OR HANDLED BY INEXPERT AND/OR UNAUTHORIZED PERSONNEL FOR ANY REASON.

NOTA

THE INFORMATION IN THIS MANUAL IS SUBJECT TO CHANGE WITHOUT NOTICE. M.O.COM. SRL SHALL NOT BE RESPONSIBLE FOR DIRECT, INDIRECT, ACCIDENTAL OR CONSEQUENTIAL DAMAGE OR OTHER DAMAGES RELATED TO THE SUPPLY/USE OF THIS INFORMATION. THIS DOCUMENT MAY NOT BE REPRODUCED, ADAPTED OR TRANSLATED, IN WHOLE OR IN PART, WITHOUT THE PRIOR, WRITTEN AUTHORIZATION OF M.O.COM. SRL
mocom IS REGISTERED TRADEMARKS OF M.O.COM. SRL

MANUAL SCOPE

The purpose of this manual is to provide instructions for:

- becoming generally familiar with the product;
- its correct installation;
- loading the materials and for safe and efficient use of the equipment.

Moreover you will find:

- general technical features of the product;
- procedure to access the menu and set the parameters;
- maintenance procedures;
- troubleshooting and problem solutions;
- other useful documentation.

GENERAL WARNINGS

The product must **always** be used according to the procedures included in this manual and never for purposes other than the ones indicated.

WARNING

THE USER IS RESPONSIBLE FOR ALL LEGAL REQUIREMENTS RELATED TO THE INSTALLATION AND USE OF THIS PRODUCT. THE MANUFACTURER WILL NOT BE RESPONSIBLE FOR ANY BREAKAGE, MALFUNCTIONS, PROPERTY DAMAGE OR INJURY IN THE EVENT THAT THE PRODUCT IS NOT INSTALLED OR USED CORRECTLY.

Please observe the following precautions in order to avoid injury or property damage; in particular:

- **ONLY** use recommended detergents and chemical additives. The use of other products could damage the unit.

WARNING

THE USE OF DETERGENTS OR CHEMICAL ADDITIVES DIFFERENT FROM THE ONES RECOMMENDED COULD LEAD TO A SERIOUS DAMAGE OF THE UNIT. SEE APPENDIX A.

- **Do not** pour water or other liquids or inflammable products on the equipment;
- **Do not** use the equipment in the presence of gas or explosive or inflammable vapors;

WARNING



WHENEVER IT IS NOT POSSIBLE TO DISCONNECT THE ELECTRICITY TO THE DEVICE, OR IF THE EXTERNAL POWER GRID SWITCH IS FAR AWAY OR, AT ANY RATE, NOT VISIBLE TO THE MAINTAINER, PLACE A WORK IN PROGRESS SIGN ON THE EXTERNAL POWER GRID SWITCH AFTER TURNING IT OFF .

- Make sure the electrical system is **grounded** conforming to current laws and/or standards;
- **Do not remove** any label or nameplate from the device; request new ones, if necessary.
- **Do not** lean on the door and do not use it as a step.
- **Only use original replacement parts.**

WARNING

THE FAILURE TO OBSERVE THE ABOVE, RELEASES THE MANUFACTURER FROM ALL LIABILITY.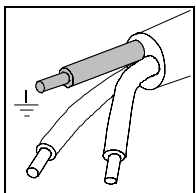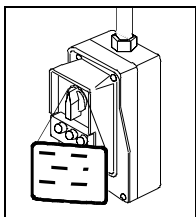


CONTENTS OF THE PACKAGE

DIMENSIONS AND WEIGHT

Package

Overall weight ca. 52 kg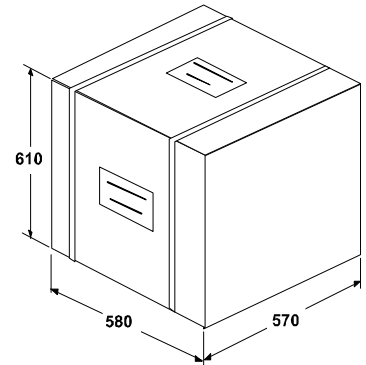

NOTE

CHECK THE INTEGRITY OF THE PACKAGE UPON RECEIPT.

Once the package is opened, check that:

- the supply matches the specifications of the order (see the accompanying document);
- that there is no obvious product damage;


NOTE

IN THE CASE OF A WRONG PRODUCT, MISSING PARTS OR ANY TYPE OF DAMAGE, IMMEDIATELY PROVIDE A DETAILED DESCRIPTION TO THE RESELLER AND THE TRANSPORTER THAT MADE THE DELIVERY.

ENGLISH

DESCRIPTION OF THE CONTENTS

- * **millwashing** equipment;
- * Basket for trays and with hand-piece/pipe cleaning facility (Ref. 1);
- * Vertically arranged instrument tray (Ref. 2);
- * 8-position tray holder (Ref. 3) ;
- * Basket for millers (Ref. 4);
- * Bottle with detergent (Ref. 5)
- * Water drain and supply pipes (Ref. 6 and 7)
- * Bag containing hand-piece support adapters and fitting for pipe cleaning (Ref. 8)
- * Bag containing the draining pipe installation kit (Ref. 9)
- * Operator's manual / Warranty certificate (Ref. 10)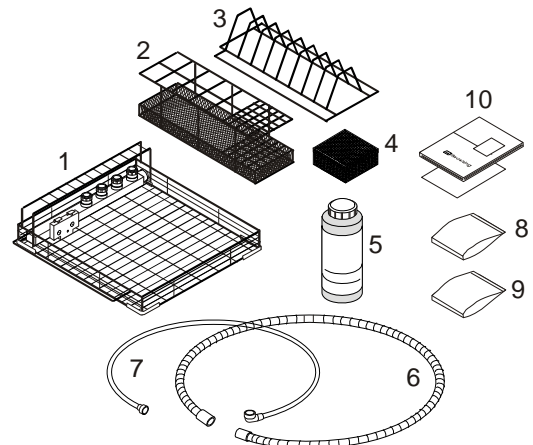

NOTE

THE CUSTOMER MUST KEEP THE WARRANTY CERTIFICATE TOGETHER WITH THE RECEIPT.

LIFTING AND TRANSPORT

Where possible, the packaged product must be handled using suitable mechanical means (forklift truck, transpallet, etc.) and following the instructions shown on the package.

In the case of manual handling, the product must be lifted by two persons using the handles cut in the side of the box. Once removed from the box, the unit must be lifted by two persons and transported on a cart or other similar device.

WARNING

WE RECOMMEND THAT THE DEVICE BE TRANSPORTED AND STORED AT A TEMPERATURE NO LOWER THAN 5 °C. PROLONGED EXPOSURE TO LOW TEMPERATURE AN DAMAGE THE PRODUCT.

NOTE

KEEP THE ORIGINAL PACKAGING AND USE IT WHENEVER THE DEVICE IS TO BE TRANSPORTED.

WARNING

BEFORE TRANSPORT, DAIN THE WATER CONTAINED, LEAVE THE DEVICE TURNED-OFF FOR ABOUT 30 MINUTES SO THAT ALL THE HOT INTERNAL PARTS WILL HAVE TIME TO COOL.

PRODUCT INFORMATION

INTRODUCTION

millwashing is an instrument washer and thermo-disinfector specially designed for use in dental surgeries.

It provides optimal performances, cycles for efficient treatment of solid materials, and ease of use.

It represents an interesting solution for all professionals that require equipment with favourable price/quality and performance ratio.

GENERAL FEATURES

millwashing is an electronically controlled unit managed by a microprocessor.

It consists of a cleansing compartment with a door provided with a sealing gasket, washing detergent tank, washing pump, detergent pump, drain pump, filters within the internal plumbing, a set of electro magnetic valves for the hydraulic operation and a coil for the steam condensation. A control panel allows to select the cycles and set a series of operating parameters; LCD display shows alarm messages for a quick troubleshooting.

The unit is equipped with a water softener to reduce the quantity of scale contained in the water network. The softening rate can be programmed according to the water hardness.

Thanks to the electronic management and the integrated sensors, thermostats and pressure switch, an high stability of thermodynamic parameters is obtained and consequently an optimal process control. An internal memory stores the data and events related to a maximum of 4000 cycles performed.


millwashing is equipped with effective safety devices to preserve the user from injuries due to malfunction and anomalies.

NOTE


PLEASE REFER TO APPENDIX A (TECHNICAL CHARACTERISTICS) FOR A DESCRIPTION OF THE SAFETY DEVICES.

FRONT VIEW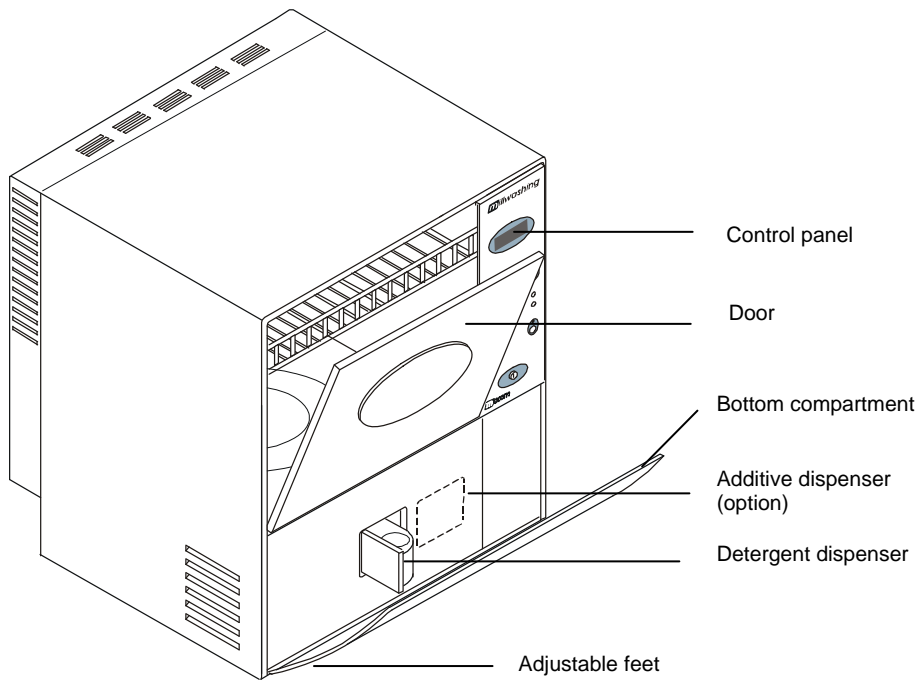


REAR VIEW

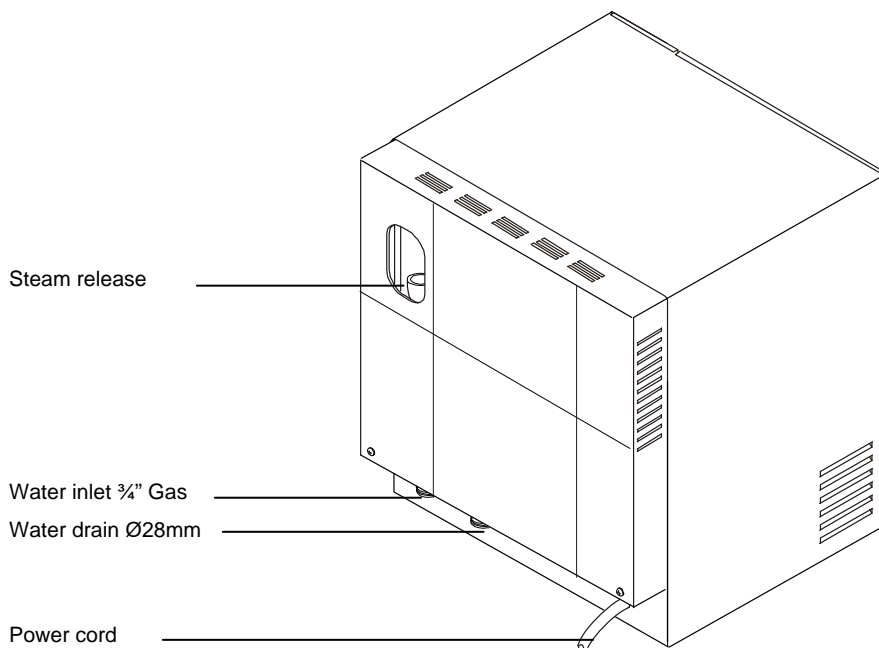ENGLISH

CONTROL PANEL

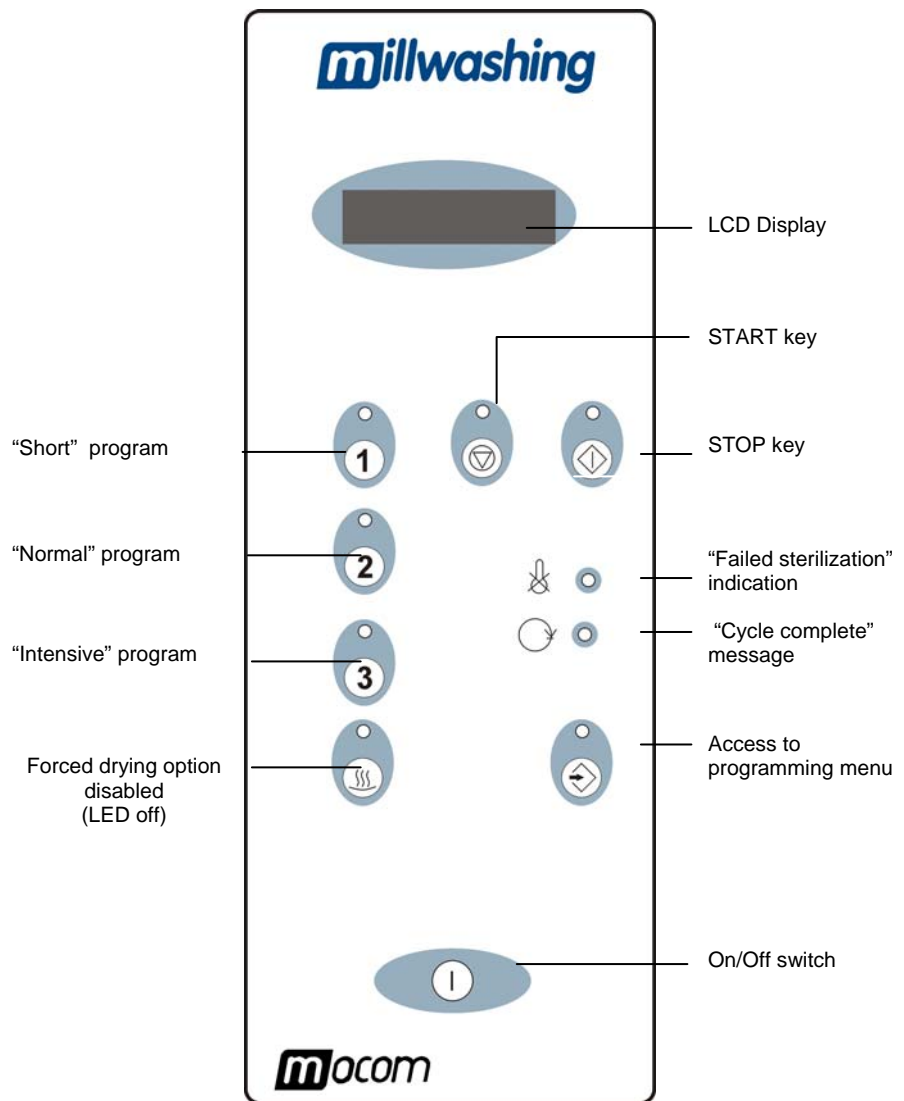Display Two-lines display showing data and messages.
 In the standby status it displays the temperature value.
 After the Start, it displays the current temperature value.
 In "Block" status, it displays the alarm message relating the type of fault.

Buzzer Continuous tone upon pressure of any key in the control panel, and intermittent tone in Block status.

**INSTALLATION
INTRODUCTION**

The first and fundamental step in achieving good processing, long life and complete use of its features is a correct, careful installation. Moreover, this precaution will avoid the danger of physical injury or property damage, not to mention malfunctions and damage to the machine. So, please follow the instructions in this chapter **scrupulously**.

WARNING

INSTALLATION MUST BE PERFORMED BY QUALIFIED PERSONNEL.

NOTE


M.O.COM CUSTOMER SUPPORT. (SEE APPENDIX Z) IS AVAILABLE TO CLEAR ANY DOUBT OR SUPPLY FURTHER INFORMATION. THE UNIT IS SUPPLIED AS TESTED AND AFTER HAVING PASSED ALL THE SCHEDULED FACTORY TESTS.

ENGLISH

Size and weight

- Height (total) 555 mm
- Length (total) 560 mm
- Depth 600 mm
- Overall weight about 44 kg

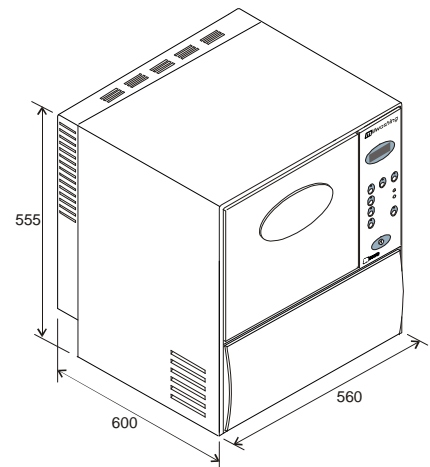Power supply

The electrical system to which the sterilizer will be connected must be suitably dimensioned based on the electrical characteristics of the device. This information is shown on the **back of the machine**.

**GENERAL
PRECAUTIONS
FOR THE
INSTALLATION**

- Install the unit on a plane strong enough to support its weight (44 kg) and at a height of at least 25 cm; if necessary adjust the feet to compensate any plane inclination.

WARNING

LEAVE ADEQUATE SPACE FOR VENTILATION (AT LEAST 10 CM ON EACH SIDE) ALL AROUND THE UNIT, ESPECIALLY IN THE BACK AND UPPER TO ALLOW ITS CORRECT VENTILATION.

- Do not install the sterilizer near tubs, sinks or similar places, to avoid contact with water or liquids. This could cause short circuits and/or potentially dangerous situations for the operator;
- Do not install the sterilizer in a place that is excessively humid or poorly ventilated;
- Do not install the machine where there is gas or inflammable and/or explosive vapors;
- Install the unit so that the power cord and the inlet/outlet pipes are not bent or pressed.

WARNING


STEAM OUTLET FROM THE BACK. INSTALL THE UNIT SO THAT THE BACK FACES A WALL, NOT A SPACE FOR STANDING OR PASSING BY.

COMPARTMENT DIMENSIONS FOR BUILT-IN INSTALLATIONS

When installing the unit inside a cabinet, you must provide adequate space all around the device to provide effective ventilation as well as a large enough opening in the back that, in addition to allowing the passage of the power cord and inlet/outlet pipes, will also provide an adequate air flow and the consequent optimum cooling of the heat exchanger.

It is indispensable that the built-in compartment have the following **minimum dimensions** shown in the figure.

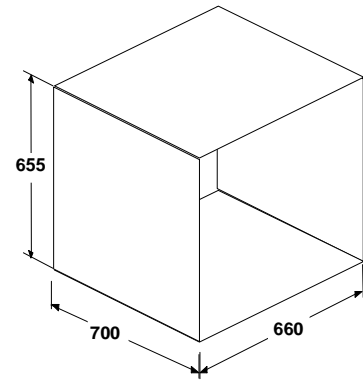- Height 655 mm
- Width 660 mm
- Depth 700 mm

WARNING

COMPARTMENT DIMENSIONS LESS THAN THOSE SHOWN MAY COMPROMISE THE CORRECT CIRCULATION OF AIR AROUND THE DEVICE AND MAY NOT PROVIDE ADEQUATE COOLING, WITH THE CONSEQUENT DETERIORATION OF PERFORMANCE AND/OR POSSIBLE DAMAGE.

WARNING

DO NOT REMOVE THE UPPER COVER OR ANY OTHER EXTERNAL PART. WHEN INSTALLED IN THE COMPARTMENT, THE DEVICE MUST BE COMPLETE WITH ALL ITS PARTS.

NOTE

PLEASE REFER TO APPENDIX A (TECHNICAL CHARACTERISTICS) FOR COMPLETE TECHNICAL DATA.

ELECTRICAL CONNECTION

The sterilizer's must be connected to a socket of the electrical installation of adequate capacity for the device's absorption, and ground provided, in conformity with current laws and/or standards.

WARNING

ARRANGE A POWER SUPPLY LINE EQUIPPED WITH A DIFFERENTIAL BREAKER AND AUTOMATIC SWITCH WITH CONTACTS OPENING OF AT LEAST 3mm. THE ELECTRICAL INSTALLATION MUST BE GROUNDED IN COMPLIANCE WITH THE STANDING NORMS.

The differential and automatic breakers should have the following features:

- Nominal current I_n 16 A
- Differential current $I_{\Delta n}$ 0,03 A

NOTE

THE MANUFACTURER WILL NOT BE LIABLE IN CASE OF INSTALLATION OF THE EQUIPMENT WITH INADEQUATE ELECTRICAL FACILITIES AND/OR NOT SUITABLY GROUNDED.

DO NOT USE MAINS EXTENSIONS, ADAPTORS OR OTHER ACCESSORIES ARRANGE THE ELECTRICAL INSTALLATION SO THAT THE SWITCHES CAN BE EASILY REACHED.

HYDRAULIC CONNECTION

The unit should be connected to the water outlet and drain inlet points in compliance with the standing norms.

Check that the water pressure is within the range shown in the table.

Min/Max Pressure			
Pressure	Min kPascal (bar)		Max kPascal (bar)
Static	250	(2.5)	500 (5.0)
Dynamic	200	(2.0)	400 (4.0)

- If dynamic pressure is lower than 2 bar (200 kPa) install a pump to increase the pressure.
- If the pressure is higher than 5 bar (500 kPa) install a pressure reducer.

WARNING

FOR WATER HARDNESS HIGHER THAN 10 °FR A SOFTENER DEVICE SHOULD BE MOUNTED UPSTREAM.


Water supply connection

Install a tap provided with ring nut threaded 3/4" GAS and placed so that it can be easily accessible.

NOTE

AN ANTI-REFLUX DEVICE IS ALREADY INSTALLED ON THE EQUIPMENT WATER INLET.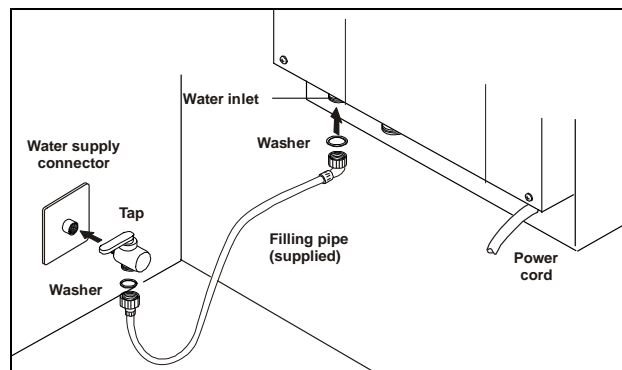

Use the supplied pipe to connect the adapter threaded 3/4" GAS of the equipment to the ring nut of the tap.

WARNING

DO NOT SHORTEN OR DAMAGE THE PIPE.

Water drain connection


WARNING

THE CENTRAL DRAIN MUST BE SUITABLE TO RECEIVE WATER AT HIGH TEMPERATURE CONTAINING CHEMICAL AND BIOLOGICAL SUBSTANCES.

The machine is equipped with a threaded connector Ø 28mm for the water drain.

Mount the supplied ring nut and the pipe-union on the threaded connector and seal with the washer.
Use the supplied draining pipe to connect both pipe-unions (side equipment and central drain); tight with clamps.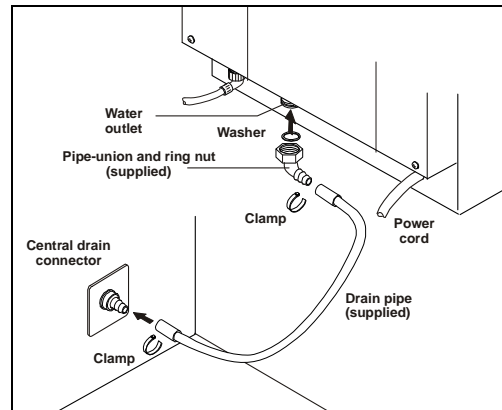

EMBEDDED WATER SOFTENER

The embedded water softener is made up of resin filters that reduce the salinity of the water supplying the machine.

In case of particularly hard water, the unit deteriorates quickly affecting its performance and life.

In order to keep active, it is necessary to periodically regenerated the resins, especially when the display shows the message “**SALT LOAD**” or “**CLN**” (see *Appendix C “Maintenance”*)

In the units equipped with this device, the installer should program, according to the water hardness, a special parameter for showing the regeneration message as necessary.

FILLING THE DETERGENT

The compartment under the door includes one 500 cc dispenser for the liquid detergent and a pump to dose the detergent ; the dose value is programmable and preset according to the data of the detergent manufacturer (average consumption about 18 cc. per cycle).

WARNING

TO ENSURE THE CORRECT OPERATION OF THE PUMPS, HAVE THE GENERAL MAINTENANCE CARRIED OUT AS PER THE SCHEDULE INCLUDED IN APPENDIX C.

Open the door of the lower compartment and fill the container with the detergent.

NOTE

**IT IS RECOMMENDED TO USE THE DETERGENT “MEDICLEAN” BY THE DR.WEIGHERT COMPANY.
SEE APPENDIX A” – TECHNICAL CHARACTERISTICS.
FOR MORE DETAILS CALL THE CUSTOMER SUPPORT OR THE RESELLER.**

The units signals the lack of detergent by a message on the display. For further details see Appendix “E”.

NOTE

THE UNIT CARRIES OUT THE CYCLE ALSO WITHOUT DETERGENT.

**FILLING THE
ADDITIVE (OPTION)**

The compartment under the door includes an additional 500 cc dispenser (optional) for a liquid additive and a pump to dose it ; the dose value is programmable and preset according to the data of the additive manufacturer.

<u>WARNING</u>	
	TO ENSURE THE CORRECT OPERATION OF THE PUMPS, HAVE THE GENERAL MAINTENANCE CARRIED OUT AS PER THE SCHEDULE INCLUDED IN APPENDIX C.

Open the door of the lower compartment and fill the dispenser with liquid additive.

<u>NOTE</u>	
	IT IS RECOMMENDED TO USE THE CHEMICAL ADDITIVE “MEDIKLAR” BY THE DR. WEIGHERT COMPANY. SEE APPENDIX A” – TECHNICAL CHARACTERISTICS. FOR MORE DETAILS CALL THE CUSTOMER SUPPORT OR THE RESELLER.

The units signals the lack of chemical additive by a message on the display. For further details see Appendix “E”.

<u>NOTE</u>	
	THE UNIT CARRIES OUT THE CYCLE ALSO WITHOUT CHEMICAL ADDITIVE.

ENGLISH

OPERATION

MACHINE STATUSES

The unit can take the following statuses:

- STANDBY:** The unit is ready to start a cycle.
Self diagnostics are active.
- CYCLE:** The unit enters in this status with the **Start** command that is only accepted for machine in standby and door closed.
The cycle starts and the various phases are successively executed. Self diagnostics and controls are active. The display shows the current value of the temperature.
- BLOCK:** The internal diagnostics has detected an anomaly involving the block of the operation; the cycle is interrupted and the door remains blocked. The anomaly is signaled with the red Led ( ), and the machine is waiting for the door unblock procedure allowing the return in Standby state (see door unblocking procedure).

LOAD THE MATERIALS

Insert the instruments to be washed by carefully positioning them in the special holders and tray.

- The objects must not overlapped.
- Position the objects so as to avoid deposit of water inside or on its external surface.
- Arrange high and heavy objects in the centre of the basket to facilitate the washing.

WARNING

ARRANGE THE OBJECTS SO THAT THEY DO NOT BLOCK THE WASHING ARMS; CHECK THAT THEY CAN TURN FREELY.



NOTE

THE MAXIMUM OVERALL LOAD ALLOWED FOR EACH CYCLE IS 8.0 KG.

Handpieces and turbines

The washing and thermo-disinfection of hollow instruments must take place both outside and inside the cavity. To this purpose, the basket is equipped with special accessories in which the handpieces and turbines can be plugged.
For the washing of pipes, fittings of different diameter are available.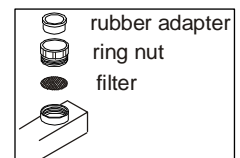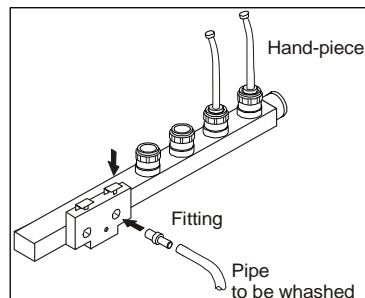

The handpiece accessories are made of a rubber adapter (different diameters available), ring nut and filter.

NOTE

WASH THESE FILTERS WEEKLY OR REPLACE THEM.

The program for the correct washing and thermo-disinfection of straight and angled handpieces and turbines includes:

- washing at 65°C with neuter PH detergent and free of mineral substances.
- thermo-disinfection at temperature of 90°C (93° as option) for 1 minute, with the addition of additive to eliminate traces of water residue.

The suggested chemical products are:

- MEDICLEAN, neuter PH detergent.
- MEDIKLAR, additive for the elimination of water residue.

NOTE

**DO NOT SUBMIT MICROENGINES TO THERMO-DISINFECTION.
FOR HANDPIECE AND TURBINE TREATMENT, IT IS RECOMMENDED TO USE ONLY THE PROGRAM NR. 1.
DO NOT USE WASHING AND ADDITIVE POWDERS.**

CHOICE THE CYCLE

SHORT PROGRAM

Suitable for materials not very dirty.

Washing only with detergent and thermo-disinfection at 90°C (93°C as option) for 1 minute.

NORMAL PROGRAM

Suitable for materials normally dirty.

Pre-washing, washing with detergent and thermo-disinfection at 90°C (93°C as option) for 3 minutes.

INTENSIVE PROGRAM

Suitable for materials specially dirty.

Intensive pre-washing, washing with detergent and thermo-disinfection at 90°C (93°C as option) for 10 minutes

START THE CYCLE

Check for detergent in the internal container and the materials properly loaded:

1. Close the door and switch on the unit by pressing the key ,
2. Select the desired cycle (the key's Led turns on),
3. Press the **START** key  (the key's Led turns on).

The cycle starts and goes through a series of programmed phases. The display shows the current value of temperature. The door is blocked throughout the cycle.

DRYING OFF

Press this key to exclude the forced drying phase from the selected cycle.

STOP TEMPORARY AND COMPLETELY THE CYCLE

Press this key if you need to stop the cycle in execution; the key's red Led switches on.

The failed disinfection is signaled by **flashing the red Led** .

From this condition you can:

- resume the cycle from the start by issuing a new **START** command,
- or select a new **STOP** command to definitely stop the cycle; the door is unblocked and the signaling on the display goes back to the standby status.

NOTE

THE STOP KEY IS DISABLED DURING THE FINAL DRYING PHASE.

END OF CYCLE

In conditions of normal operation, the cycle continues up to the completion of any phases.

At the end of the cycle the **green Led**  turns on to signal the successful completion of the cycle.

IN CASE OF BLACK-OUT

Upon the mains restore after a black-out, the unit will signal the event by showing on the display the message **"POWER FAIL"** and store the washing program position before the black-out.

At the mains restore, the unit can enter in Standby or Block status. From this condition it is possible:

- enter a **START** command to resume the cycle;
- carry out the **unlocking procedure** and repeat the cycle.

DOOR UNBLOCKING PROCEDURE

In "Block" status, the door remains blocked; to unblock the door it is necessary to perform the following procedure:

1. Press together **START** and **STOP** keys for 5 seconds

2. Press the program selection keys in sequence

The unit enters in "Standby".

NOTE

IF THE FAULT CONDITION PERSISTS FOR A MALFUNCTION, THE DOOR IS UNBLOCKED BUT THE UNIT STILL DOES NOT WORK. IN THIS SITUATION CALL THE CUSTOMER SUPPORT.

RESINS REGENERATION

When the unit detects a value of water hardness over the threshold (degradation of the water softening function), the message **"SALT LOAD"** or **"CLN"** is shown on the display.

In this case it is necessary to carry out the resin regeneration procedure. See Appendix "C" - Maintenance.

APPENDIX A – TECHNICAL CHARACTERISTICS

SUMMARY TABLE

Unit	Instruments washer and thermo-disinfector
Model	
Manufacturer	M.O.COM. Srl Via Saliceto, 22 40013 Castel Maggiore (BO) Italy
Power supply voltage	220/230 Vac (other voltages upon request)
Mains frequency	50 Hz
Dimensions (L x D x H)	560 x 600 x 555 mm (with closed door)
Nominal power	2750 W
Insulation class	Class I
Installation category	Cat. II
Environment of use	Indoors
Environmental operating conditions	Temperature: +15°C ÷ +35°C Relative humidity: 80% (max), non condensing Altitude: 2000 m upon sea level.
Net weight	about 44 kg
Dimensions of the chamber (W x D x H)	400 x 470 x 260 mm
Max load of materials	8 kg
Pressure of incoming water	Min 250 kPa (2,5 bar) – static pressure 200 kPa (2,0 bar) – dynamic pressure Max 500 kPa (5,0 bar) – static pressure 400 kPa (4,0 bar) – dynamic pressure
Available cycles	3
Water softener (resins)	Embedded (regeneration to be carried out by user) Programmable water softening rate (according to water hardness)
Detergent dispenser	500 cc
Additive dispenser (option)	500 cc
Detergent consumption	18 cc/cycle (average value) – configurable value
Additive consumption	3 cc/cycle (average value) – configurable value

ENGLISH

SAFETY DEVICES

The unit is equipped with the following safety devices for which we provide a brief description of their function:

- **Fuses protecting the electronic circuits**
Protection against a fault in the low voltage devices and circuits.
Action: cuts power to one or more low-voltage circuits.
- **Safety thermostat on the heating resistor**
Protection against overheating of the heating resistor.
Action: interrupts the electrical power supply of the resistor.
- **Safety thermostat on the drying resistor**
Protection against overheating of the drying resistor.
Action: interrupts the electrical power supply of the resistor.
- **Door blocking mechanism**
Protection against accidental door opening (also due to a black-out).
Action: it prevents accidental opening of the door during a cycle.
- **Monitoring of the equipment operation**
Real time monitoring of all significant parameters when the machine is powered.
Action: generation of alarm messages (in the case of anomaly) with possible interruption of the cycle.

RECOMMENDED DETERGENTS AND ADDITIVES

“MEDICLEAN” detergent (DR. WEIGHERT) or other type with equivalent characteristics (ex. DECONEX 23 NEUTRAZYM - Borer Chemie AG).

MEDIKLAR Additive (DR. WEIGHERT) or other type with equivalent characteristics, to eliminate traces of water residue.

The suggested products has been used to carry out the performance tests on the unit.

- Only use recommended chemical detergents and additives. The use of other products could damage the machine.

NOTE

THE USE OF RECOMMENDED CHEMICAL ADDITIVES DOES NOT INVOLVE MANUFACTURER’S LIABILITY FOR DAMAGES THAT COULD OCCUR ON TREATED MATERIALS AND OBJECTS.

- Follow the indication provided by the manufacturer of the chemical products.

WARNINGS ABOUT THE CHEMICAL PRODUCTS

The unit was designed to operate with water and chemical additives. Do not use organic solvents or of other types.

DANGER

DANGER OF EXPLOSION OR FAST DETERIORATION OF A FEW COMPONENTS OF THE MACHINE

- Solvent or acid residues, especially the “hydrogen chloride”, damage the steel; avoid the contact.
- Never use washing or additive powders.
- Never use foam detergents.
- Do not use chemicals based on chloride (chlorine bleach, sodium hypochlorite, hydrogen chloride etc..). These chemicals irreparably damage the unit and affect the integrity of the instrument.

APPENDIX B – PROGRAMMING

ACCESSING THE MENU

To access the setup Menu, keep pressed for 5 seconds the key . The display shows the item **Menu**.

In programming phase, use:

- key to scroll the menu options and the parameters within the option,
- key to increase the parameter value shown on the second line of the display
- key to decrease its value.

Set the date and time

Date and time should be set to get the correct data history recording in the unit's history.

Action	Description	Display (two-lines messages)
Press repeatedly	Displays the Menu options up to date setting option	Set date
Press the key	Access to the parameter Time	Clock Current Time
Press the key or	The current time is modified	Clock Modified time
Press the key	Access to the parameter Minutes	Clock Current Minutes
Press the key or	The current minutes are modified	Clock Changed minutes
Press the key	Access to the parameter Month date	Date Current month date
Press the key or	The month date is modified	Date Changed month date
Press the key	Access to the parameter Week day	Data Current week day
Press the key or	The week day parameter is modified	Date Current week day
Press the key	Access to the parameter Month	Date Current month
Press the key or	The month is modified	Date Modified month
Press the key	Access to the parameter Year	Date Current year
Press the key or	The year is modified	Date Modified year
Press the key	Access to the option to confirm the entered modifications	Date Confirm mod ?
Press the key	The modifications are confirmed	Data Modifications OK
Press the key	Return to the Menu	Menu

Select the language for the displayed messages

Action	Description	Display (two-lines messages)
Press repeatedly 	Displays the Menu options up to the language selection option	Language
Press the key 	Access to the currently selected language	Language Italian
Press the key 	Change of language for displayed messages	Language English
Press the key 	Stores the selected language and goes back to Menu	Menu

Print the stored reports

Action	Description	Display (two-lines messages)
Press repeatedly 	Displays the Menu options up to the print reports option	Print reports
Press the key 	Prints all the stored cycles	Print To quit: Stop

NOTE

AT THE END OF THE PRINTING, THE UNIT AUTOMATICALLY GOES BACK TO MENU.

Description of the fields in the Events History

The microprocessor records and stores the data of the performed cycles and the possible fault events. These data are arranged in the following fields:

DATE	START TIME	PROGRAM	MAX °C	HOLD>85°C	FAULTS
dd-mm-yy	12.00	Short	90°C (93°C as option)	60 seconds	00
dd-mm-yy	13.05	Normal	90°C (93°C as option)	180 seconds	00

The memory capacity allows the data recording of maximum 4000 cycles. Reaching the 90% of available capacity, the display will show a proper message (TRANSFER DATA). Carry out the print operation and delete the history archive to allow new records.

Faults occurred during the cycles are shown in the field FAULTS with specific numeric codes (see *Appendix "D" – Alarm messages*):

Deleting the stored Events History

Action	Description	Display (two-lines display)
Press repeatedly 	Displays the Menu options up to history clear option	Delete History
Press the key 	Password is requested to carry out the clearing action. See NOTE	Password
Press the key 	Return to the Menu	Menu

NOTE

TO CARRY OUT THE CLEARING OF THE EVENTS HISTORY, CONTACT THE CUSTOMER SUPPORT OF THE RESELLER THAT SUPPLIED THE PRODUCT.

OTHER PARAMETERS CAN BE PROGRAMMED BY THE SERVICE IF NEEDED.

APPENDIX C – MAINTENANCE

ORDINARY MAINTENANCE PROGRAM

The table summarizes the maintenance required to keep the sterilizer operating at peak efficiency.

In the case of very intense use, we recommend shortening maintenance intervals:


DAILY	- Cleaning of external surfaces
WEEKLY	- Nozzle cleaning - Cleaning of internal surfaces and accessories - Disinfection of external surfaces - Cleaning or replacement of the handpiece holder
WHEN REQUIRED	- Regeneration of the water softener
YEARLY	- General revision by the Service

MAINTENANCE BY THE OPERATOR

General warnings

Keep the following general warnings in mind

- **Do not** wash the unit with direct jets of water, either under pressure or sprinkled. Seepage into electrical and electronic components could damage the functioning of the device or its internal parts, even irreparably;
- **Do not** use abrasive cloths, metal brushes (or other aggressive materials) or products for metal cleaning, whether solids or liquids;
- **Do not** use chemicals products or disinfectants. In fact these products may damage internal components, even irreparably.
- **Do not** allow lime residue or other substances to accumulate in the tank, on the door or its gasket; periodically remove any residues. In fact, they can damage these parts over time in addition to compromising the operation of the components installed along the plumbing circuit.

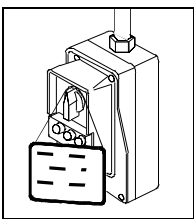DANGER

BEFORE PERFORMING ORDINARY MAINTENANCE, MAKE SURE THAT THE STERILIZER'S MAIN SWITCH IS TURNED OFF. WHENEVER IT IS NOT POSSIBLE TO DISCONNECT THE ELECTRICITY TO THE DEVICE, OR IF THE EXTERNAL POWER GRID SWITCH IS FAR AWAY OR, AT ANY RATE, NOT VISIBLE TO THE MAINTAINER, PLACE A WORK IN PROGRESS SIGN ON THE EXTERNAL POWER GRID SWITCH AFTER TURNING IT OFF

NOTE

IN CASE OF VERY HARD WATER, PERIODICALLY CARRY OUT A GENERAL REVISION AND CLEANING OF THE UNIT. PAY ATTENTION TO THE INTERNAL RESISTORS AND TO THE THERMOSTAT'S BULBS.

Cleaning the external surfaces

Clean all external surfaces with a clean cloth dampened with water, possibly adding weak neuter detergent.

Wipe all surfaces and remove any residue before using the unit again.

Cleaning the control panel


Clean the control panel with a soft clean cloth dampened with a product for plastic or other disinfectant suitable for this use.

Cleaning the nozzles

To eliminate scale traces from the nozzles of the rotating arms, use a clean cotton cloth soaked with a weak solution of water and vinegar (or similar product, after first checking its contents on the label).

- Remove the washing arms.
- To make the cleaning of the arms inside easier, unscrew and remove the two caps placed at their ends.
- Remove any scale residue.

ENGLISH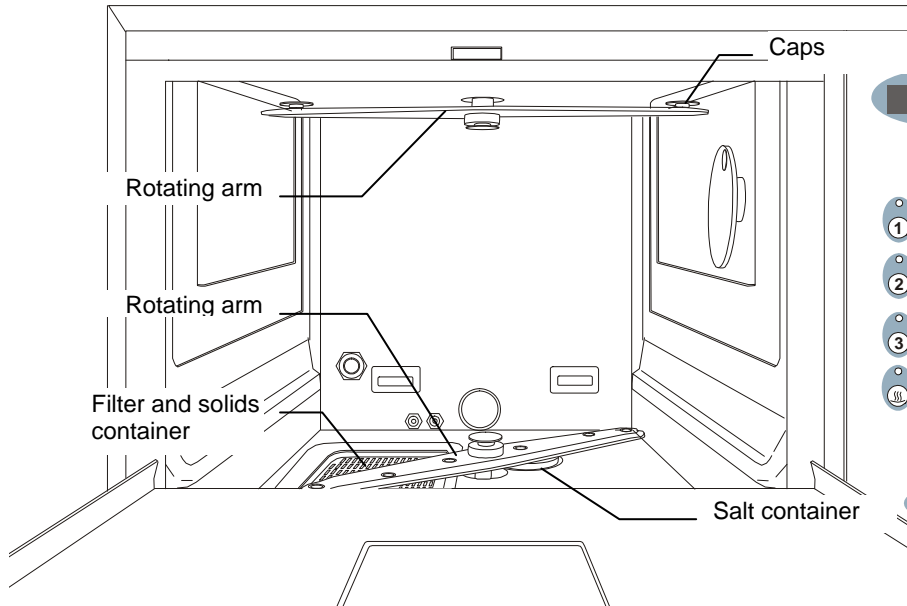


Disinfecting the external surfaces

To disinfect external surfaces use denatured alcohol or products composed with a minimum percentage of sodium hypochlorite.

Cleaning/replacing the handpiece and accessory holder

Separate the parts that make up the handpiece holders and place all the accessories in a container filled with a weak solution of water and vinegar. Let the solution dissolve the scale deposits. Wash under current water and dry.

Replace the internal filter of the handpiece holder if it is no longer recoverable.

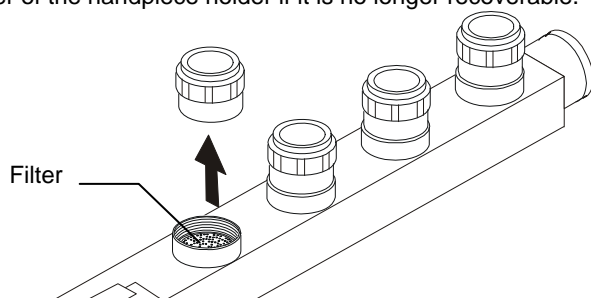Regenerating the water softener

When the display shows the message “SALT LOAD” or “CLN” it is necessary to load salt into the unit; proceed as follows:

- Open the washing compartment door.
- Unscrew the plastic cap of the salt container placed inside.
- Pour about 0.5Kg of common kitchen salt into the container using the funnel provided.

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- Screw the plastic cap back.
- Carry out a normal load of material to be washed and start a normal cycle.

The unit will automatically perform the regeneration of the water softener.

NOTE

THE FIRST WASHING CYCLE STARTED AFTER THE REGENERATION IS LONGER AND APPARENTLY PRESENTS AN INITIAL IDLE STAGE DURING WHICH THE UNIT PERFORMS THE REGENERATION PROCESS. DURING THIS PHASE THE DISPLAY SHOWS THE MESSAGE “REG”.

MAINTENANCE BY THE SERVICE

To ensure the perfect operation of the pumps dosing the chemical products, it is recommended to have them maintained every three months.

Every 12 months

Request a general maintenance that will include among other things:

- Cleaning and if necessary replacement of the internal filters;
- Cleaning and if necessary replacement of the electromagnetic valve's membranes;
- Cleaning of the pump to ensure its perfect rotation;
- Control and if necessary cleaning of the thermostat's probes;
- Replacement of the dosage pump's membrane.

NOTE

THIS GENERAL MAINTENANCE IS NECESSARY ALSO FOR INSTALLATION USING SOFT WATERS; THE HIGH WATER TEMPERATURE INSIDE THE MACHINE FACILITATES THE DEPOSIT OF SCALE ON THE HEATING RESISTORS AND THE OBSTRUCTION OF THE NOZZLES SO AFFECTING THE WASHING AND THE ACHIEVEMENT OF THE DISINFECTION TEMPERATURE IN THE TANK.

DISPOSAL AT END-OF-LIFE

In accordance with Directives 2002/95/ EC, 2002/96/ EC and 2003/108/ EC, regarding the reduction in use of dangerous substances in electrical and electronic equipment, as well as waste disposal, such equipment may not be disposed of as normal urban waste and must be separated accordingly. When purchasing a new, equivalent piece of equipment, the old piece of equipment that has reached its end-of-life must be handed over to the reseller for proper disposal. As regards reuse, recycling and other forms of recovery of the above mentioned waste, the manufacturer carries out the functions defined in the individual national legislations.

The proper collection and separation of such equipment for recycling, treatment and disposal helps avoid any possible negative effects on the environment and health and facilitates the recycling of the materials of which the equipment is made. The crossed out rubbish can symbol indicates that the product, at the end-of-life, must be collected separately from other types of waste.

WARNING!

Improper disposal of the product results in the application of sanctions which are defined by individual national laws.

APPENDIX D – GENERAL PROBLEMS

INTRODUCTION

If during the use of the unit an alarm indication should appear, this does not mean that the machine is faulty and out of service.

The problem could be due just to a transitory anomaly (for example a black-out), or an improper use of the equipment.

First, it is important to identify the cause of the anomaly and work on it when possible before requesting the intervention of the **Technical support service (see Appendix Z)**.

To this end, you will find here indications for troubleshooting and solution of the most common problems, besides an accurate description of the alarms codes along with their meanings (see *Appendix E – Alarms indications*).

INTERNAL SELF DIAGNOSTICS

The unit is equipped with a microprocessor card that includes analogical and digital I/O interfaces connected to devices and probes in order to continuously monitor the values of the preset parameters and the proper operation of the unit.

No detergent/rinse-aid product:

It is detected by a special detector.

This alarm does not interrupt the cycle in progress and does not prevent the start of a cycle.

Failed resistor:

If the command of resistor enable remains active for a time longer than the preset value, the cycle is interrupted and the machine switches to blocked status.

Door blocking warning:

It intervenes when both the messages of blocked door and open door are detected by the door's micro-switches.

Water fill Time-out:

It takes over if the cold-water electromagnetic valve stays open over the time-out value or for a water level lower than the preset value after the timeout.

Water drain Time-out:

It takes over for a water residue higher than the threshold level after the drain pump timeout.

Door unblocked:

It takes over when the door blocking command sent by the electronic card does not match with the confirmation signal from the door's micro-switch within the time programmed; the machine switches to blocked status and generates the door unblocked message.

Temperature probe fault:

It takes over for a fault in the tank's or heating resistor's temperature probe; the unit switches to blocked status.

TROUBLESHOOTING AND SOLUTIONS

If the unit does not work properly, perform the following checks before contacting the Technical Support Service:

PROBLEM	POSSIBLE CAUSE	SUGGESTED SOLUTION
The unit does not switch on.	Slack power supply connections.	Check the terminals on the switchboard and restore the connections.
	No power supply voltage	Check the cause of the power failure.
	Intervention of the circuit breaker and/or differential switch of the power supply line.	Reset the switches and try again the operation.
The cycle does not start	Door open	Close tight the door.
The cycle stops and the machine switches to blocked status.	The display is showing the alarm Er 3 or Er 4	Door open . Close tight the door and repeat the cycle.
	The display is showing the alarm Er 7	No water. Check that the water tap is open. Repeat the cycle.
	The display is showing the alarm Er 0	Black-out during the cycle. Repeat the cycle.
Excessive humidity on the materials at the end of the cycle.	Excessive quantity of material in the washer.	Check the quantity of material and does not exceed the max value allowed (See Appendix A , " Technical characteristics ").
	Materials not properly placed.	Positions the materials according to indications. (See Chapter " Load of materials ").
	Obstructed filters.	Call the Technical support service (See Appendix Z , " Technical support ").

NOTE

IF THE PROBLEM PERSISTS, CALL THE TECHNICAL SUPPORT SERVICE (SEE APPENDIX Z) COMMUNICATING MODEL AND SERIAL NUMBER.
THESE INFO ARE REPORTED ON THE DATA PLATE ON THE UNIT REAR AND ON THE WARRANTY CERTIFICATE.

APPENDIX E – ALARMS INDICATIONS

INTRODUCTION

If during operation the diagnostics detects an **anomaly condition**, the unit shows an alarm code on the display.

According to the type of detected alarm, the unit immediately stops the cycle or causes the Led   of failed disinfection to flash at the end of the cycle.

MEANING OF THE ALARMS ON THE DISPLAY

Message	Meaning
Er 0 : POWER FAIL	A blackout occurred during the cycle.
Er 2 : FAILED DISINFECTION	Temperature did not reach the preset value during the disinfection phase.
Er 3 : DOOR LOCK	During the cycle the door's blocking device switches off and the door remains blocked. Or the start command and consequent enabling of the door's blocking device do not match with the blocking-performed signal from the door's micro-switch within the timeout set by the programming parameter.
Er 4 : DOOR OPEN	During the cycle the door's micro-switch detects the open status; so there is an anomaly of door blocked but open.
Er 7 : WATER LACK	Insufficient water level in the tank after the timeout set by the programming parameter. Possible causes of this block can be lack of water or the timeout set at too low value.
Er 8 : HEATING ELEMENT SAFETY	The heating resistors were inserted but the disinfection stage was not completed within the timeout set by the programming parameter. Possible causes of this block can be a failure of the resistors or the timeout set at too low value.
Er 9 : TEMPERATURE PROBE 1	Fail of the temperature probe #1 in the tank.
Er 12 : PUMP FAILURE	Insufficient pressure in the washing circuit. Problems of the pump or pressure switch.
Er 15 : DRAIN PUMP	Excessive presence of water during the drain phase.
Er 16 : CHEMICALS LACK	Lack of washing chemicals
Er 18 : DRYING FAILURE	Insufficient pressure in the forced ventilation circuit due to a failure of the fan, filter obstruction or failure of the pressure switch
Er 21 : PROBE TEMPERATURE MISMATCH	The two temperature probes read different temperatures during the disinfection phase.
Er 22 : TEMPERATURE PROBE 2	Fail of the temperature probe #2 in the tank.

NOTE

IN CASE OF VERY SHORT BLACK-OUT, THE MACHINE COULD SIGNAL ALARM **ER3 (DOOR LOCK)** INSTEAD OF **POWER FAILURE**.

IF ON THE OCCURRENCE OF ALARM **ER3** THE DOOR APPEARS UNBLOCKED AND THEREFORE CAN BE OPENED, THE CAUSE OF THE ERROR IS CERTAINLY DUE A SHORT POWER FAILURE.

THE ERROR **POWER FAILURE** IS ONLY SIGNALLED IN CASE OF BLACKOUT DURING THE CYCLE.

CODES REPORTED IN THE FIELD “FAULTS” OF THE EVENTS HISTORY

The result of the performed cycle is signalled in the field “Faults” of the events history with a numeric code that takes into account possible events occurred during the operation that caused a blocked status:

Code	Meaning
01	cycle ok

Code	Meaning
00	power fail
02	failed disinfection
03	Door unblocked
04	Door open
07	water lack
08	Resistor failure
09	probe 1 failure
10	Serial interface failure
12	Pump failure
15	drain pump failure
16	chemicals lack
18	Drying blocked <i>(for units equipped with the forced drying option)</i>
21	Probes mismatch
22	Sterilizer probe 2 failure

APPENDIX G – DECLARATION OF CONFORMITY

DICHIARAZIONE DI CONFORMITÀ DECLARATION OF CONFORMITY DECLARATION DE CONFORMITE KONFORMITÄTSBESCHEINIGUNG DECLARACION DE CONFORMIDAD

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APPENDIX Z – TECHNICAL SUPPORT

**FOR ANY REQUEST FOR
TECHNICAL SERVICE FOR THE PRODUCT,
WHETHER IN OR OUT OF WARRANTY,
DIRECTLY CONTACT THE
TECHNICAL SUPPORT DEPARTMENT
OF THE DEALER OR RESELLER
THAT SUPPLIED THE PRODUCT.**

M.O.COM. Srl is completely available to customers to provide any technical information about the product as well as to offer suggestions and advice on steam sterilization procedures.

In this regard, please refer to the following address:

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INTRODUCTION

Symboles

DIRECTIVES EUROPEENNES APPLICABLES

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(M.O.COM. S.r.l.)