



Tissue Sampling Tags


Handling & Tagging Instructions

www.BvdFreeEngland.co.uk

Dalton TAGS




1 Open bag with care
Check cutter is firmly attached to point of male tag

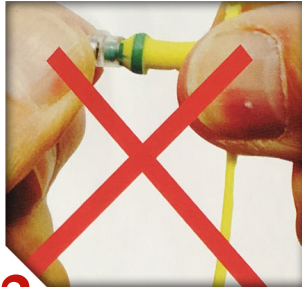

2 Do not remove cutter



3 Load female first
Insert tube and secure tag below clip

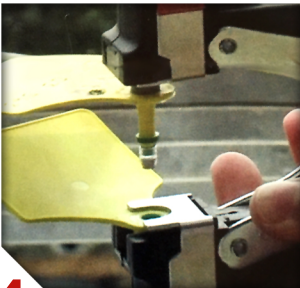

4 Load male second
push onto pin
Ensure cutter is attached



5 Insert tag between two cartilage ribs
Close applicator fully
Audible click is heard!



6 Open applicator fully to release tag



7 Remove sample tube
Push tube gently up from below



8 Check tube is sealed and contains sample



9 Sealed tissue sample has same number as the tag