



Cushion Instructions

Please read fully before using cushions

Please ensure all cushions fit correctly.

Ensure all cushions stick and strap to the chair correctly, if not do not use.

- This can be checked by pushing the cushion on to the chair then pull the cushion up to ensure suction is occurring. Pull tabs on suckers to release.
- If straps are fitted, Adjust strap before closing the buckle and ensure straps are tight.

The manufactures lap strap should be used at all times in conjunction with the chair cushion.

Only use cushions for intended use and specified type of bath and chair.


Do not stand or walk on cushions

Do not use cushions in conjunction with any products which may contain oils as this may make the cushion slippery.

Frail or assisted bathers should not be left alone when using bath cushions.

If a cushion is damaged do not use it.

If in doubt about any aspect of a cushion or its application or fitting, contact your supplier before use.

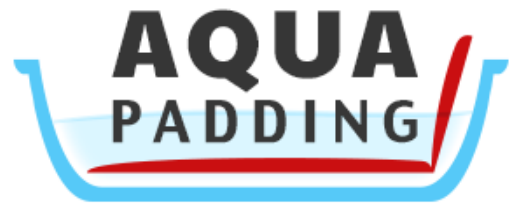

Cleaning and Care

- Remove cushion from bath after use.
- To remove cushions release one sucker at a time starting at a corner.
- Release suckers by lifting the tab on each sucker. Do not pull on the cushion. This will invalidate your warranty.
- Clean with any detergent not containing bleach
- Aqua Padding can recommend a detergent please contact us for further information www.aquapadding.co.uk
- Wipe down front and rear of cushion to dry it.
- Do not place cushions near naked flames or heat sources.
- Do not use sharp implements or abrasive cleaners on cushions.
- Only use sponges and /or soft cloths to clean and dry cushions.
- When not in use store cushions flat.

Please make these instructions easily and readily available for all personnel to access before using cushions.